

Minivator Bathing Solutions

Bringing bath time back





Bliss

“Perfect happiness,
great joy”

Oxford English Dictionary

Many people experience difficulty in using their bath either as a result of illness or limitations to their mobility. If you are having difficulty with getting in and out of the bath you may be considering using one of a number of products designed to make bathing safer and easier.

If you are looking to enjoy the benefits of a soak in the bath without the disruption or cost of having your bathroom adapted you may find using a bath lift is the answer. Most bath lifts are effectively a seat that can be placed securely in the bath when needed. When raised to its upright position it should be level with the top of your bath. You can then sit on your bath lift and move your feet and legs over into the bath. You then lower the seat into the bath, with the water already in it, and regain the pleasure of a relaxing bath. When you are finished the bath lift is raised to the starting position and you can exit the bath without having to run the water out first. When not in use your bath lift can be removed from the bath leaving it free for other users.

A bath lift is often preferable to a shower (and shower seat) because it offers you the



The Bath Bliss Reclining Bath Lift lowers you gently into the bath and gives you a choice of three final recline positions: 10°, 35° or 45°.

muscle relaxing benefits that a soak in the bath has to offer.

Bath lifts are also often a preferred option over a walk in bath because the nature of such a bath is that you would need to get in when empty and wait for it to fill up. You then need to wait until the water has drained away before opening the door to exit the bath. This can cause discomfort as the user may get cold during either waiting period. The installation of these units may also be quite disruptive and could leave your bath room in need of redecoration.

Minivator Bath Bliss Reclining Bath Lift



As the name suggests, the Minivator Bath Bliss from Handicare will

recline, allowing you to lie back in the bath.

This means that your bath lift will need to be placed away from the end of the bath so there is room for it to recline. It is easily operated and provides a safe and comfortable way of enjoying a bath.





The benefits of a Minivator Bath Bliss

The team at Handicare, formerly Minivator, has worked hard to establish an international reputation as manufacturers of safe, reliable quality lifting products. Our bath lifts have been designed with the user in mind and we are confident that they will be suitable for use in most types of bath.

In choosing Handicare you will be choosing a well made bath lift created to be practical whilst also looking stylish. Compliant with international safety standards, Handicare bath lifts have also undergone a rigorous testing regime so your bath lift will come with peace of mind that you have a bath lift you can trust.

All Handicare bath lifts come with an ergonomic waterproof handset that is simple to use. Up and down are clearly

marked, textured finishes and large buttons also make them easy to use for those with visual impairments.

Stability in the bath is provided thanks to the super sucker system which securely anchors your bath lift in place, the superior six sucker design provides even greater stability for baths with anti-slip surfaces.

Recognising that it is important the suckers grip well but also need to be easy to remove, we have included an integral quick release design. The handle on the bath lift's frame makes it easy to remove your bath lift leaving the bath clear for other users or for cleaning.

Your Minivator Bath Bliss from Handicare also has machine washable comfort covers and can be used with bubble bath, bath salts or oil based bathing products.



Easy to handle

The unique, patented frame can lift the unit easily without the need for tools when handled.

Easy to use

To operate your bath lift lightly depress the button for the chosen direction of travel. To stop the bath lift at any time release the button.

The Minivator Bath Bliss will recline to your pre set final recline position once it has reached the end of its downward travel.





Easy to remove

Simply pull up on the release handle to release the front super suckers, then tilt the lift back to release the rear suckers and remove, leaving the bath clear for other users.


The Minivator Bath Bliss divides into two pieces, each of which can be handled separately.

lip means that you can
frame opening up



Easy to maintain

The Minivator Bath Bliss can be cleaned with non abrasive cleaners and the comfort covers can be removed for machine washing up to 60°C.

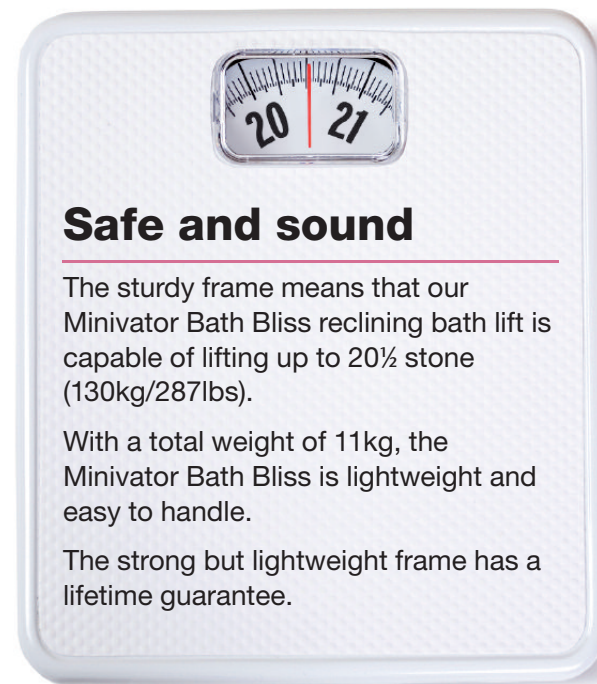

Safe and sound

The sturdy frame means that our Minivator Bath Bliss reclining bath lift is capable of lifting up to 20½ stone (130kg/287lbs).

With a total weight of 11kg, the Minivator Bath Bliss is lightweight and easy to handle.

The strong but lightweight frame has a lifetime guarantee.

Easy to bath again

Handicare bath lifts bring a relaxing bath time back for people who find it difficult or painful to get in or out of their bath.

Available from mobility dealers nationwide they fit into most baths to provide a safe and reliable means of getting in and out of your bath once more.



Swivel and Slide

Added mobility
for greater comfort



Available in blue or white, Swivel and Slide Boards are designed to further aid users who may have difficulty turning their body or who require assisted bathing.

They are placed on the bath lift seat to assist users in moving across the bath lift to a forward facing position, or in turning to exit the bath.

Perfectly suited to your needs



The Minivator Bath Bliss will lower you straight up and down in the bath and recline when the seat reaches the bottom of the bath.

It gives a choice of three final recline positions: 10°, 35° and 45°. You can easily determine which of these positions is used when fitting the back rest to the seat base. This position is set each time the back is fitted allowing you to change the recline position to suit different users.


Technical data

Weight capacity	287lb (20.5 stones)	130kg
Weight of unit	24.2lbs	11kg
Weight of the heaviest part	15.2lbs	6.9kg
Maximum seat height (A)	17.0ins	431mm
Minimum seat height (floor to seat when seat is in its lowest position)	3.15ins	80mm
Maximum overall height of unit (B)	39.5ins	1005mm
Length of unit in upright position (C)	26.4ins	670mm
Backrest to front edge of seat (D)	19.0ins	482mm
Maximum width of backrest (E)	13.9ins	352mm
Seat to top of backrest (F)	22.9ins	581mm
Width of seat excluding flaps (G)	15.0ins	380mm
Width of seat with side flaps fully extended (H)	26.5ins	672mm

HMV number: 04.40.01.0xxx

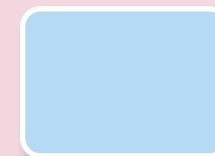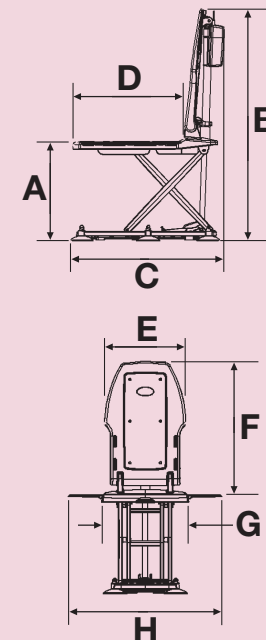

Aegean
covers

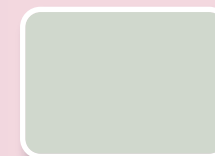

Silver Sage
covers

Your questions answered

This section answers many popular questions about bath lifts. If you have any further questions please contact your Handicare dealer, or healthcare professional. Either will be happy to help.



If my bath lift is battery powered how can I use it in water?

Your Minivator Bath Bliss is built to comply with international safety and quality standards. The battery is sealed away in the waterproof handset so you have no need to worry about using this powered product with water.

To recharge you simply unplug the handset and use the charging unit provided (like a mobile phone charger) to charge it in another room.



How do I know the battery will not run out while I am in the bath?

Your Minivator Bath Bliss handset will have an easy to read visual warning so you can see if the battery needs recharging. When the light on the handset is amber it is time to recharge.

You do not need to worry about getting stuck in the bath. Your Minivator Bath Bliss has an intelligent system built into its handset. It knows if it does not have enough power to lift you back out of the water so it will not lower you in the first place.



As the seat is made of plastic is it comfortable to sit on?

Your Minivator Bath Bliss comes with anti-slip comfort covers as standard so you can be sure of a comfortable bathing experience. Not only will they help prevent you slipping when the seat is in the water, they also make it warmer to sit on when you are about to use it.

The covers can be removed for machine washing at 60°C.



Is it complicated to use?

The handset for your Minivator Bath Bliss has two large buttons, one up and one down. These are clearly marked and have tactile features that assist users with visual impairments to distinguish between them.

To move the lift in either direction hold the appropriate button down. If you let the button go, the lift will stop until either button is depressed again. There is also an emergency override button for additional peace of mind.





Handicare Accessibility Ltd
82 First Avenue
Pensnett Estate
Kingswinford
West Midlands DY6 7FJ
England

User support: 0870 118282
Global Dealer Support: +44 (0)844 225 3121
Email: enquiries@handicare.co.uk

www.handicare.co.uk

North America
10 Industrial Park Drive
Mount Pocono PA
18344
United States

Tel: 570 895 2480
Toll Free: 1 866 276 LIFT/5438
Email: info@sterling-stairlifts.com

www.sterling-stairlifts.com

Germany
Handicare Treppenlift
Gesellenweg 7
32427 Minden
Germany

Tel: +49 (0)571 973398-0
Fax: +49 (0)571 973398-33
Email: treppenlifte@handicare.de

www.handicare.de

International website

www.handicare.com

Also available from Handicare

Straight and Curved
Stairlift Systems to
suit most staircases



Please be aware that actual colours may vary slightly from those shown here. All information is provided in good faith but does not constitute a guarantee. Handicare reserves the right to alter designs and specifications without prior notice.

HC0810

