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- Lightweight, durable, rust-resistant frame
- Wide seat with handles provides additional comfort
- Available with and without backrest
- Seat height adjusts in 1" increments for various user heights
- 350 lb weight capacity

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© 2008, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360. Made in China. [www.grahamfield.com](http://www.grahamfield.com)

Specifications subject to change without notice  
Especificaciones son sujetas a cambio sin aviso

GF0900003RevB09

## Instructions for Use – Please Read All Instructions Before Use.

### Setup

#### Installing the Back

(Only for 7921A-4)

1. Place bath seat on its side on a flat working surface.
2. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
3. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

NOTE: Ensure seat support tube buttons protrude fully through channel guide holes.

Caution: Backrest is not designed to support the full weight of the user.

### Adjusting Leg Height

Legs are already installed on the bath seat.

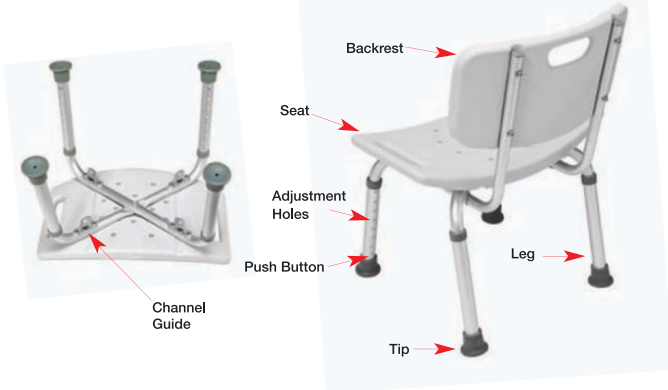
To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.

NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.

2. Repeat STEPS 1 and 2 for the other three legs.

NOTE: Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.

## Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at [www.grahamfield.com](http://www.grahamfield.com).

## Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

## WARNING

**WARNING:** DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** Bath Seats 7931A & 7921A-4 has a maximum user weight limitation of 350 lbs.

**WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**WARNING:** ALWAYS inspect the bath seat to ensure that it is properly positioned and stable before using. DO NOT use if bath seat is wobbly or unstable.

**WARNING:** If components are damaged or missing, contact your GF Health Products, Inc. Health Products distributor immediately, DO NOT use substitute parts.

**WARNING:** Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips IMMEDIATELY.

**WARNING:** DO NOT use this as a transfer bench or climbing device.

**WARNING:** All four leg tips MUST be in contact with shower/tub floor at all times.

**WARNING:** Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes BEFORE use.

**WARNING:** Only use GF Health Products, Inc. accessories with this product.

**WARNING:** This product should only be used with tub floors wider than 18 ¾ inches.

**WARNING:** Users with limited physical capabilities must be supervised or assisted when using the bath seat.

**WARNING:** Make certain ALL screws and nuts are tight at all times.

**WARNING:** DO NOT submerge product in water, otherwise damage may result.

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### User Instructions and Warranty Information

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## Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex models listed on the cover of this hangtag for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care.

Parts covered under the lifetime warranty include: frame, welding, screws, and frame locking components. Specific items “excluded” from this warranty of any kind are: “ALL” other parts not specifically listed above. The warranty does not extend to non-durable components, such as tips, which are subject to normal wear and require periodic replacement. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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**ADVERTENCIA:** Utilice solamente accesorios de GF Health Products, Inc. con este producto.