Breezy 100

Gebrauchsanweisung
Directions for use
Notice D'Utilisation
Gebruikershandleiding
Manuale Per l'Uso
Manual De Utilização
Manual De Usuario

BREEZY

INNOVATE.
BUILD FOR THE FUTURE

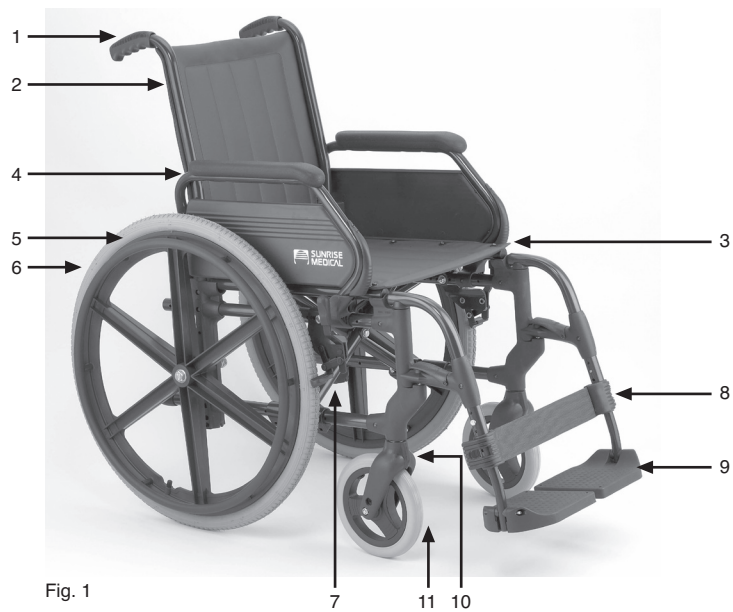
Fig. 1

Fig. 2

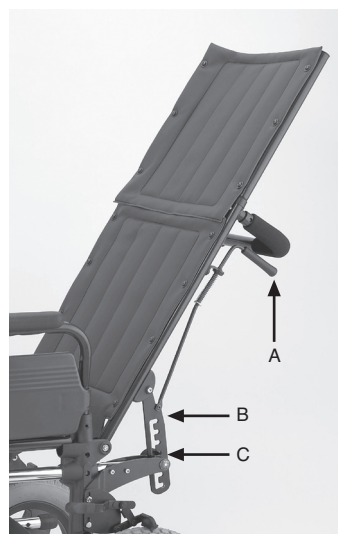
Fig. 3

Fig. 4

Fig. 5

Fig. 6

Fig. 7

Fig. 8

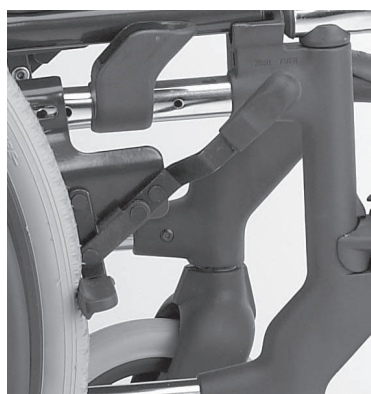
Fig. 9

EINLEITUNG

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INTRODUCTION

Thank you for choosing a SUNRISE MEDICAL product. In view of the broad range of components and accessories available for all SUNRISE MEDICAL wheelchairs it is in your interest to consult a health professional or authorised SUNRISE MEDICAL distributor. These wheelchairs are designed for persons with a physical handicap, to be used only for transport. These wheelchairs are suitable for use both indoors and outdoors (Class B product). The manufacturer reserves the right to modify, without prior notice, weights, dimensions, or any other technical features described in this manual. All figures, measures, and capacities included in this manual are approximate and are not specifications.

Models: 105, 112, 115, 110, 121, 141

Dealer stamp

Sunrise Medical is ISO 9001 certified, which ensures quality at all stages of the development, and production of this wheelchair.
This product is manufactured to comply with the radio interference requirements of EEC directive 89/336/EEC Medical Device Directive 93/42/EEC.

INDEX

A.CHARACTERISTICS	13	F. CONTROLS AND MAINTENANCE	19
B.SAFETY	13	G.SPECI FICA TIONS	20
C. HANDLING	16	H. MEASURES.....	20
D. ASSEM BLY	17	I. GUARANTEE	20
E. ADJUSTMENTS.....	19		

A. CHARACTERISTICS

- | | |
|-----------------------|---------------|
| 1. Handgrip | 7. Wheel lock |
| 2. Back upholstery | 8. Footrest |
| 3. Seat upholstery | 9. Footplate |
| 4. Armrest upholstery | 10. Fork |
| 5. Push rim | 11. Castor |
| 6. Propulsion wheel | |

B. SAFETY TIPS

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed.

- ☐ Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents.
- ☐ As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. You should always wear clothing which is light-coloured when it is dark, so that you can be seen more easily. Also, make sure that the reflectors are not covered.
- ☐ Be careful during your first ride in this wheelchair. Get to know your wheelchair. Test what effect changing the centre of gravity would have on your wheelchair. Practice using your wheelchair on level ground as well as going uphill and downhill.
- ☐ CAUTION: The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always use both wheel locks; otherwise, your wheelchair could tip over. The wheel locks

have not been designed to be used as brakes for a moving wheelchair. **Warning: do not put your fingers into the braking system.**

- ❑ Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants. They should grip the wheelchair at securely mounted parts. A wheelchair should never be lifted with an occupant; it should only be pushed.
- ❑ Before each use, the following should be checked: Quick-release axles (if fitted) on the rear wheels / Tyres, tyre pressure, and wheel locks. Wheel locks will not function properly if tyre pressure is too low or the interval between tyres and wheel locks is too large.
- ❑ The maximum user weight is 125 kilograms. Never overload the chair by carrying rucksacks, etc. or a child on your lap.
- ❑ Do not attempt to climb or descend gradients of more than 10% with the maximum weight of 125 kg. Always lean your body towards the top of the slope to correct for changes in the centre of gravity.
- ❑ If the chair has been left in the sun for a long time parts such as the fabric, armrest padding and platforms may burn the skin on contact.
- ❑ It is dangerous to tip the chair back so that the front wheels are off the ground: it could overturn and injure you. Do not attempt to do "wheelies" unless you are an experienced user or are accompanied.
- ❑ When mounting kerbs and other obstacles, ensure that the anti-tipping wheels are facing backwards so that they do not become caught on the kerb.
- ❑ If the push rim is damaged replace it at once: otherwise splinters and chips could injure your hand when you push.

TRANSPORTATION IN VEHICLES

This product is suitable to use as a seat in vehicles as per certificate **ISO/FDIS 7176-19 :2001 (E)** and using the restraint system **Q strain Q-5001-T2**.

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security of a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances where the user must be transported whilst in the wheelchair the following advice should be followed.

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheel chair and accessories.
2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs) meeting the requirements of ISO 10542 or SAE J2249, in accordance with the WTORS manufacturers' instructions.
4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances. (Fig. 10)

Warning There is a risk of serious injury or death if this advice is ignored.

5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.
6. The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.
8. Alterations or substitutions must not be made to the wheelchair securement points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.
9. Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig. 11) The upper torso restraint belt should be mounted to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

Fig.10

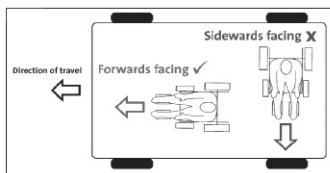
Fig. 11


10. A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.
11. Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19:2001 or SAE J2249.
12. The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.
13. Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, for example:
Crutches
Loose cushions
Tray Tables
14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.
15. Reclining backrests should be returned to an upright position.
16. The manual brakes must be firmly applied.

Occupant Restraint Instruction

1. The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal.
A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75degrees.
(Fig 12)

Fig.12

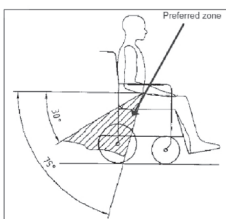
2. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig 13 and Fig 14.
Restraint belts must be adjusted as tightly as possible consistent with user comfort.
Restraint belt webbing must not be twisted when in use.
The upper torso restraint belt must fit over the shoulder and across the shoulder as illustrated in Fig 13 and Fig 14.
3. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Fig.13

Fig. 14

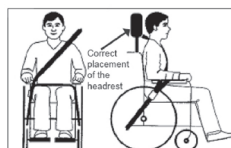4. The attachment points to the chair are the inner front side frame just above the castor and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes. (See Figs 15-20)
5. The tie down symbol (Fig 16) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

Transportability – positioning of wheelchair tie down restraints on wheelchair .

Fig. 15

Fig. 16 Tie down label

The wheelchair secured with front and rear wheelchair tie down restraints. Positioning of the restraint straps is shown in more detail below.

Fig. 17

Fig. 18

Position of the front wheelchair tie down restraint and the tie down label.

Fig. 19

Fig. 20

Position of the rear wheelchair tie down restraint and the tie down label on the wheelchair.

C. HANDLING

1. Opening: Rest both hands on each of the seat upholstery support rails and press down until the seat and back upholsteries become taut. Lower the footplates into the using position. As a safety measure, the footplates will always be in an upright position before sitting or getting out of the chair in a forward movement. **Warning: Before sitting down ensure that the seat rails are properly housed in the front structure supports. Warning: Do not place your fingers between the seat upholstery and the seat tubes when opening or folding the wheelchair. Never open the chair by pushing on the push hands.**

2. Folding: First flip up the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the out-side and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing. **Warning: Do not put your fingers between the seat upholstery and the tubing of the chair when opening or closing it.**

3. Folding backrests (Ref. 115 and 110) (Fig 1)

To fold the backrest, pull upwards the locking piece (A) and then pull downwards the back rest. To assemble: Pull upwards the back rest and pull downwards the plastic piece (A) to lock it and to secure it in this position. **Warning: Be careful not to catch your fingers when folding down or raising the backrest.**

4. Reclining backrest (Refs. 121, 141) (fig.2)

Lowering the backrest: Loosen the screw knobs (C) at the bottom of the racks. Lift the two handgrips (A) upwards at the same time, while keeping the chair steady by its handles. This movement releases the lower racks (B) from their housings leaving the backrest free to be reclined. Hold the backrest firmly to prevent it from dropping suddenly. If this should happen, quickly let go of the handgrips (A) so that the racks fall into place and prevent it from falling. With the handgrips (A) in an upward position, adjust the backrest to the desired angle using the racks' notches. Once this angle has been obtained, ensure that the racks have slipped into their notches (this operation should be done to prevent accidents caused by a sudden drop of the backrest). Release the handgrips (A). Tighten the screw at the bottom of the racks.

Raising the backrest: Proceed in the same way as for reclining the back rest.

Warning: Be careful not to catch your fingers when folding down or raising the backrest.

Warning: After the third the posterior stability of the chair is reduced, so proceed carefully.

Warning: Before raising the backrest, be sure that the user's hands or arms are not between the seat and the reclined backrest, to avoid the risk of trapping them when raising the backrest.

Warning: The reclining back is only to be used in combination with the active anti-tips setting

Warning: The reclining back is only to be used in combination with a stabilizer bar. Please always make sure that the stabilizer bar is locked in place on both sides.

Warning: Before folding the wheelchair, make sure that the stabilizer bar is removed

Warning: When adjusting the back angle, be careful not to get your fingers caught.

Warning: Before adjusting the back angle, put the anti tip tubes in to the active position

Warning: Never travel in the chair with the back-rest reclined.

5. Elevating legrest (Fig. 3) **To elevate:** Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place. **To lower:** Lift your lower leg and press the release button to unlock the legrest to lower it. As soon as you release the button, the legrest will click back into place.

Warning: The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the casters.

Warning: Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

D. ASSEMBLY

1. Footrests (Fig. 4): The footrests can be rotated inwards or outwards by pressing the release latch and turning the footrest in the desired position. To remove the footrest, press the release latch, rotate the footrest approximately 90° and lift.

2. Armrests (Fig. 5): The armrest can either be flipped back or can be detached from the chair. To flip arms back, press the release latch and pull arm toward back of the wheelchair. To detach the armrest press the button located on the back of the armrest and lift it. **Warning: Make sure that the armrest is properly locked into place. Never lift the wheelchair by the armrests as these parts are detachable and lifting the wheelchair by them may cause damage to the chair and/or injury to the user. Warning: Be careful not to catch your fingers when lowering the arm-rest.**

3. Head-rest (Fig. 6): To install the head-rest, push the head-rest tubes into the tubes at the top of the back-rest until the locking pins click into position in the holes. To remove it, press the locking pins and pull the head-rest up until it comes off.

4. Reclining back-rest stabilising bar (Fig. 6): **Installation:** Insert the bar into the tubes on the back of the back-rest. Press the locking pins and hold them down while inserting the stabilising bar, until the pins lock into place in the holes in the bar. To remove the bar, reverse the process.

5. Lap belt

⚠ Before using your wheelchair ensure the seat belt is worn and correctly adjusted before use.

The lap belt is fitted to the wheelchair as shown in the illustrations. The lap belt is fitted between the armrest support and the backrest tube. The belt should then be run across the outside of the back upholstery.

⚠ It is important that the belt is routed around the back of the wheelchair. The buckle end can be fed between the armrest and backrest.

Adjust the belt position so that the buckles are in the centre of the seat.

Adjust the lap belt to suit the user's needs as follows:

To increase the belt length:


Feed the belt through slide adjusters and male buckle to provide more belt length

To reduce the belt length:

Feed the belt back through male buckle and slide adjusters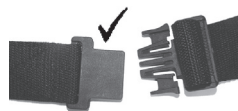


Ensure the belt is not looped at the male buckle

When fastened check space between the lap belt and the user, when correctly adjusted it should be possible to insert the flat of the hand between the lap belt and the user.


Generally the Lap Belt should be fixed so that the straps sit at an angle of approximately 45°, and when correctly adjusted should not allow user to slip down in the seat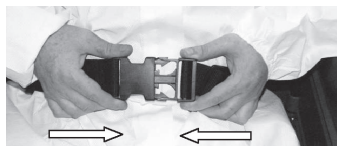

To fasten buckle:

Firmly push the male buckle into female buckle.

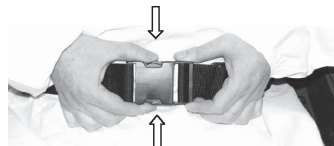To release belt:

Press the exposed sides of the male buckle and push towards the centre whilst gently pulling apart

⚠ Do not rely on the lap belt only when the wheelchair and occupant are transported in a vehicle, use the separate occupant lap and diagonal restraints provided in the vehicle..

Advice to Client

The lap belt must be checked on a daily basis to ensure it is adjusted correctly and it is free from any obstruction or adverse wear..

⚠ Failure to make sure that the lap belt is secure and adjusted prior to use could cause serious injury to the user. E.g. too loose a strap may allow the user to slip down in the chair and risk suffocation.

Maintenance

Check lap belt, and securing components; at regular intervals for any sign of fraying or damage. Replace if necessary. Clean the lap belt with warm soapy water and allow to dry.

Note

The lap belt should be adjusted to suit the end user as detailed above

Sunrise Medical also recommend that the length and fit of the belt is checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length

⚠ If in doubt about the use and operation of the seat belt then ask your healthcare professional, wheelchair dealer, carer or attendant for assistance.

6. Transport Kit

E. ADJUSTMENTS

1. Footrest heights (Fig. 7) To adjust the height of the footplates, loosen the retaining bolt in the frame tube, reposition footplate and tighten bolt. For these adjustment use a 10 mm. wrench. A minimum interval of 2.5 centimetres from the ground should always be maintained.

2. Wheel locks (Fig. 8) For a slight adjustment, loosen the nut of the screw on the clamp, using a 10 mm wrench. Slide the whole wheel lock forwards or backwards, through the groove of the clamp. For a larger adjustment, remove the screw and reset the wheel lock in the appropriate hole in the frame. Fix the new position with the screw and nut.

F. CONTROLS AND MAINTENANCE

MAINTENANCE: All fixings must be checked, adjusted and replaced when necessary throughout the useful lifetime of the chair. In particular, check the front and rear wheels, brakes, hangers, armrests and upholstery regularly. In addition to the maintenance and checks, general cleaning of both frame and upholstery can be undertaken. To maintain its appearance the upholstery should be cleaned regularly. Light soiling can be removed by sponging with a mild detergent and warm water, wiping in the direction of the texture. Rinse off with clean water before allowing to dry. Do not put upholstery in a dryer. If dirt is ingrained, the surface should be scrubbed with a soft brush. The frame should be cleaned regularly with a non-abrasive cleaner followed by polishing with a non-abrasive automotive wax.

CONTROLS: A periodic check of the following procedures will help sustaining the life and efficiency of your wheelchair.

Weekly: Check the pressure of the wheels following the instructions indicated on them. Check that the cross brace's central screw is tight and in good condition / Clean the upholstery with soapy water or adequate products / Clean the metallic parts of the chair with a dry cloth or with special products for chromium plated or painted surfaces / Check the wheel locks and adjust if necessary / Check wear of front and rear tyres.

Quarterly: Check and adjust if necessary: Castor play / Play in the castor forks / Push wheels' play / Offcentering of the push wheels / Lubricate with industrial oil or grease all parts subject to friction or wear / Retighten screws and nuts / A

general revision of the chair's condition, repairing whatever is necessary.

Yearly : We recommend a complete revision once a year in a specialized establishment

G. SPECIFICATIONS

Footrests: 70° Swing away, detachable and height adjusting. Elevating leg-rests optional

Armrests: Detachable and flip back

Standard Castors: 200 mm solid

Standard Rear Wheels: 600mm solid in models: 105, 115 and 121. 300 mm. solid in models 112, 110 and 141

Wheel Locks: high push-to-lock.

Spare Parts: Each wheelchair component is available as an spare part.

Accessories : **Footrests:** elevating. **Wheels:** Pneumatic tyres. **Wheel locks:** extension handles and drum brake. **Others:** Hemiplegic system, oxygen holder, belt, anti-tip tubes, table, crutch and dripping device supports... .. **Warning: Never use accessories not designed for this chair.**

H. STANDARD MEASURES

Model	Seat width	Seat height	Seat depth	Backrest height	Overall width	Overall length	Weight
105	37-46	51	41	45	55-64	105	17,4
112	37-46	51	41	45	51-60	95	16
115	37-46	51	41	45	55-64	105	17,9
110	37-46	51	41	45	51-60	95	16,5
121	37-46	51	41	75	55-64	115	20,6
141	37-46	51	41	75	51-60	106	18,6

I. GUARANTEE

SUNRISE MEDICAL guarantees this product for two years from the date indicated on the invoice, receipt or delivery note. For products containing batteries, the battery warranty is 6 months. This warranty covers parts in need of replacement, labour and transport.

In warranty claims, the user must submit the product and the purchase invoice as proof of delivery date.

This warranty does not cover:

- NORMAL WEAR AND TEAR, e.g. on upholstery, tyres, inner tubes, rims, wheels, fabric, etc.
- ACCIDENTS e.g. bent structures, scratches, flat tyres, etc.
- IMPROPER USE and damage caused by users who weigh more than the weight limit indicated in the user's manual.
- LACK OF MAINTENANCE and failure to follow the storage instructions in the user's manual.
- MODIFICATIONS or REPAIRS carried out by persons not authorised by Sunrise Medical.
- PRODUCTS WITH NO INVOICE OR SERIAL NUMBER (when appropriate).

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Sunrise Medical B.V.
Sunrise Medical Srl.